

PLEASE READ BEFORE PUTTING WHINNY WARMERS® ON YOUR HORSE!

1. WHINNY WARMERS ARE VERY LONG SOCKS AND GO ON MUCH EASIER IF YOU FOLD THEM NEARLY IN HALF BEFORE PULLING THEM ONTO THE LEG. ([SEE THE ENCLOSED PHOTO INSTRUCTION SHEET.](#))
2. PUT THE ENCLOSED PLASTIC BAG OVER THE HOOF.
 - * *Plastic bags make putting the socks on easy and protect the sock construction. The antimicrobial yarn is the white yarn lining inside the sock. Using the plastic bag protects this very valuable aspect of your socks.*
3. PULL THE FOLDED SOCK ON OVER THE PLASTIC BAG. REMOVE THE PLASTIC BAG FROM THE HOOF, AND PUT THE HOOF BACK ON THE GROUND.
4. ADJUST THE SOCKS UP AND DOWN ON THE LEG. **THE BOTTOM OF THE SOCK – THE PART THAT SHOULD BE AT THE HOOF – IS CLEARLY INDICATED BY RED ARROWS ON THE ENCLOSED PHOTO INSTRUCTION PAGE.**
7. REFER TO “**HOW TO REMOVE THE SOCKS**”. FOLLOW THESE INSTRUCTIONS; YOU WILL BE ABLE TO GET THE SOCKS OFF EASILY EVEN IF SOAKING WET.
8. MACHINE-WASH THEM IN WARM WATER AND DRY ON MEDIUM HEAT. DO NOT USE BLEACH. WASH THE SOCKS EVERY 3-4 DAYS TO RESTORE THE ELASTIC AND FIT.
9. **REMOVE THE SOCKS IF THEY BECOME WET.** THE IDEAL IS TO HAVE ONE SET ON AND ANOTHER CLEAN SET READY TO REPLACE THE WET ONES YOU’VE JUST TAKEN OFF.

** *As circulation-impaired or arthritic horses begin to feel better because of their warm legs, you may find the socks are not staying up as well as they did when the horse was less inclined to move due to the pain. Celebrate! If the socks are down or even come off, your horse is MOVING again! You may need to adjust them more often. In addition, in the first few days be prepared to adjust the socks more often if your horse is one that likes to mess with them. **If your horse needs more warmth on its knees or hocks, we recommend the Knee/Hock Extension Socks.***

*** **DON'T LEAVE THEM ON THE LEGS IF TEMPERATURES ARE TO GO ABOVE 50 DEGREES!** These socks are too hot for warmer temperatures.

If you have any questions, write us at info@whinnywarmers.com or call us at 850-907-5724. We enjoy talking to our customers. Go to our web site for more information: www.whinnywarmers.com.

We've never had a horse hurt or injured because of our socks. Please note that because of the theoretically possible risk of a sock coming off or falling down and panicking the horse while being ridden, we must state that riding your horse while it's wearing Whinny Warmers® must be at your own risk.