

PLEASE READ THESE INSTRUCTIONS AND HELPFUL INFORMATION.

- High-performance, embedded antimicrobial silver in the yarn continuously protect Silver Whinnys® from playing host to bacteria, fungus and mold, in the toughest conditions. The strength of the silver will not wash away or wear off. If your horse is under veterinarian care, continue to follow their instructions including the use of recommended medications. All medications that your vet may wish to use on the wound or dermatitis are fine to use with the socks and will not prevent the silver from doing its job.
 - View the video on the soxforphorses.com home page showing how to put the socks on and take them off. This is a very helpful video and will make your first experience with them a breeze. The more you practice, the easier they are to use.
 - **NEVER PUT MORE THAN 2 SOCKS ON YOUR HORSE THE FIRST 24-48 HOURS.** Silver Whinnys® feel different from anything else your horse has ever had on its legs. BEGIN IN THE MORNING WHEN YOU CAN OBSERVE YOUR HORSE'S LEVEL OF COMFORT WITH THE SOCKS. Some horses find it strange that the socks conform and move with the motion of the legs. **Make sure your horse is comfortable with the first two socks during the day light hours of the first day, and if so, allow the horse to wear the first two socks during the night. Do not add the second pair until 24-48 hours later!**
 - If you are using Silver Whinnys® as a bandage, it is important that the socks are protected and supported so that they stay in place over the wound/sores or injury. Wounds and dermatitis can be extremely painful and cause your horse to chew or rub the sox while trying to chew on its own legs. You may need to add wrap over the socks if the horse is determined to rub or chew on its legs. It is not uncommon for itching to occur due to rapid healing and equine will chew on or rub the socks down in an attempt to relieve the itching. Itching is a good thing. It indicates the condition is healing. We may recommend that your vet is called in to help with pain or itching. We might also advise a different style of sox bandage to use until your horse is no longer in pain or until the wound reaches a more stable and durable tissue surface. We are good at tech support regarding these issues. Call or email us. We can help! 850-907-5724
 - Fit is everything. Once the socks are over the hoof, they should fit the leg yet allow your hand to fit snugly between the sock and the leg. If you push the sock all the way down around the pastern and put two or three fingers between the sock and the pastern, you should not find them tight. They are made to not impair circulation even if they fall down onto the pastern. **If you are wondering if you ordered the correct size, try only one sock on your horse and observe how it fits and performs. You can always exchange them for another size.**
 - Silver Whinnys® stay up for many hours on most horses but as with any horse wear, **check them twice a day, in the morning and evening to make any needed adjustments. If you don't see your horse twice a day, make arrangements with other dependable people to check and adjust the socks for you. If you can't do this, then please don't use our socks.** For the safety of your horse, all horse clothing requires an owner or other experienced equine care persons to observe that any clothing is correctly placed, and functioning on the horse at all times. **DON'T TWIST THE SOCKS ON THE LEG. POSITION WITH THE KNIT RIBBING STRAIGHT UP AND DOWN ON THE LEG.**
 - Some equine don't like leg wear and will try to take them off. These horses may not be candidates for the socks. However, we have found that horses might accept the short socks if they didn't accept the regular length. Don't give up. Call us to discuss your horse's behavior with the socks. **Don't leave the socks on any horse that battles any horse wear.** The socks won't hurt the horse, but the horse might hurt itself if it really wants the socks off. This is why we advise that you begin in the morning with the first two socks and keep an eye on the horse during the day.
 - If it is constantly wet and muddy, change the socks with clean dry socks twice a day. The silver cannot keep up with the high level of propagating bacteria and fungi for more than 12 hours when in that environment. Otherwise, change the socks every 24 hours with clean dry socks when protection wound/sores or injuries.
- For normal wear, wash your socks every 2-3 days for best care and optimum fit. Machine wash and dry for best cleaning results. Avoid bleach, fabric softeners and perfumed detergents. Brush mud and other debris from the socks before removing them to wash. The best time to remove the socks is when they are dry. If you must remove them while wet, slide them off a bit at a time and they will come along. If muddy and wet, rinse the mud off first. **DON'T TWIST THEM ON THE LEG. KEEP THE RIB OF THE KNIT STRAIGHT.**
- We've never had a horse hurt or injured because of our socks. **Please note that because of the theoretically possible risk of a sock coming off or falling down and panicking the horse while being ridden, we must state that riding your horse while it's wearing Silver Whinnys™ must be at your own risk.** If you have any questions, call us at 850-907-5724. You can also drop us an e-mail at soxforphorses@gmail.com.
 - **If you have a horse that suffers from arthritis, laminitis, IR Cushings - Equine Metabolic Syndrome, or founder, try our new extreme temperature Whinny Warmers® with Celliant® for winter wear. Our new designs are super warm, super thick and now contain the amazing Celliant yarn. Read about Celliant at Celliant.com. Whinny Warmers® bring warmth, comfort, and circulation therapy to your horse's legs in the winter.**